



BIO3 GUIDE SURGICAL KIT[®]

for

TEMPLATE SURGERY



German Quality

CONTENT

02.....	Bio3 Guide Surgical Kit description
03.....	First Drill. Start Drills
04.....	Finish Drills
05.....	Bio3 Guide Surgical Kit product components
07.....	Additional components
08.....	Technical parameters of drill guides
09.....	Offset selection
10.....	Drilling protocol
14.....	Tools instruction for use
15.....	Notes


GUIDE SURGICAL KIT FOR TEMPLATE SURGERY



GSK

Bio3 Guide Surgical Kit (GSK) is a set of drills and instruments required for implant placement during elective surgery, using specialized software and a template. The navigation template is modeled in the software environment based on computed tomography images and 3D scans.

START DRILLS

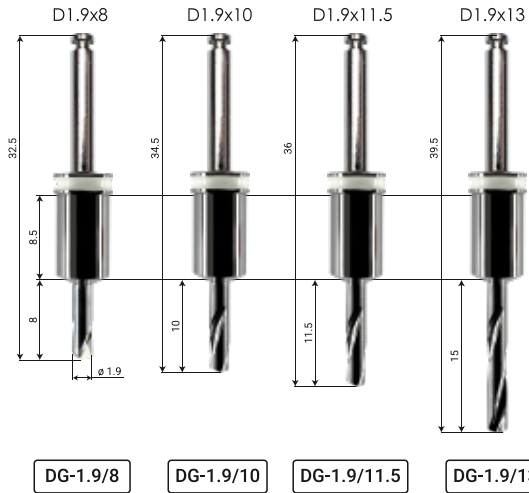

Start Drill D1.9 – Starting drills are used after the First Drill to set the depth and direction for subsequent drilling. Drill D1.9x13 has a total length of 15 mm.








DG-1.9/6

First Drill D1.9 L6 – This is the first drill used to form the initial opening in the bone and set the direction for subsequent drilling.

FINISH DRILLS

Finish Drills are used for the final formation of the bone bed for the implant.

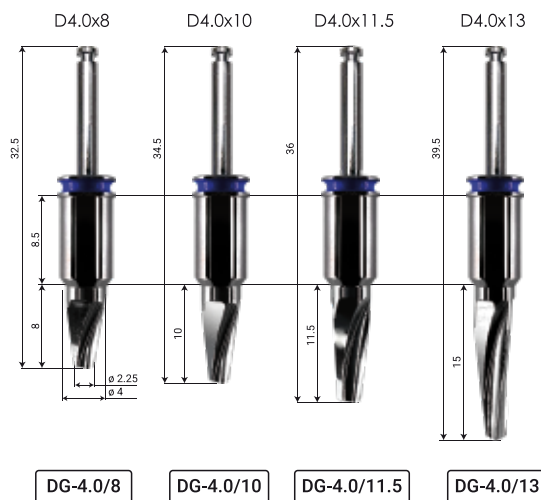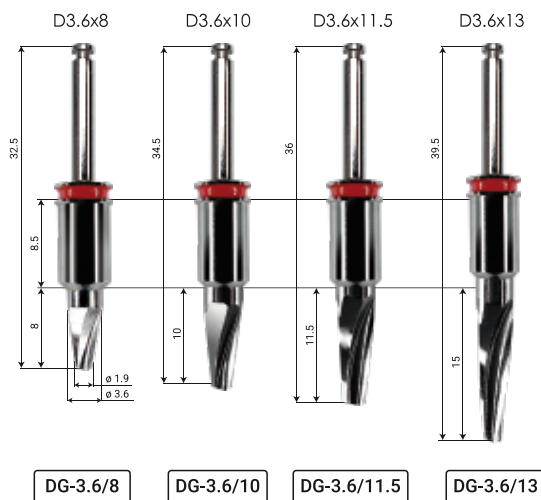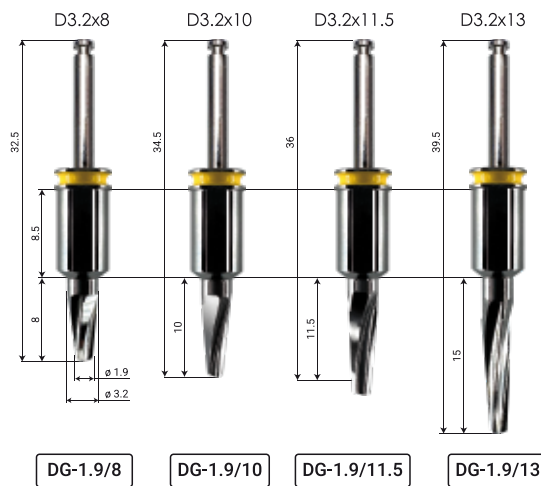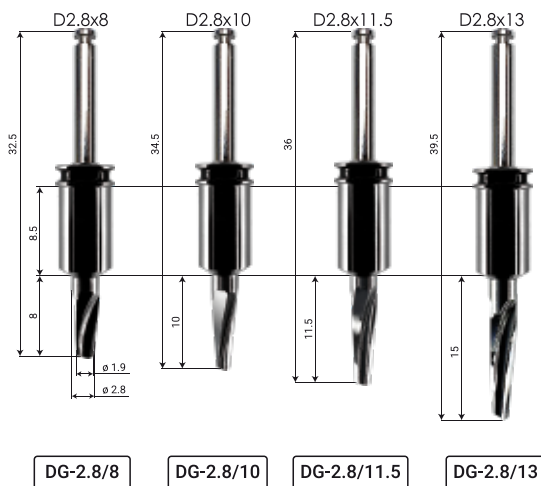

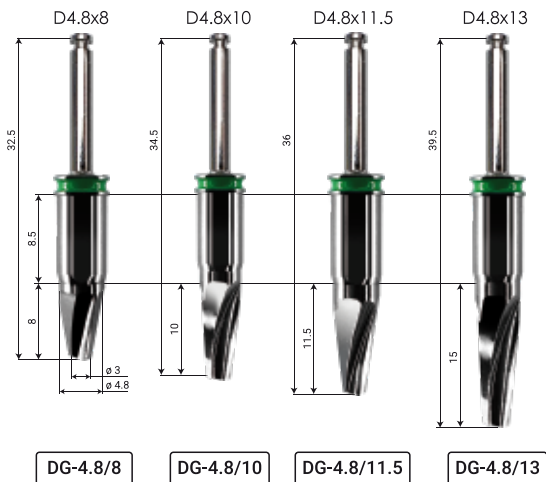

Tissue Punch is a mucotome used to perforate soft tissue through a navigation template. Tissue Punch has a surgical handpiece connection.



GS5/TP5.0(TP5.0)





MD Hex.4

Mount Driver is used for connecting Mounter Platform with a surgical handpiece.

Bio3 GUIDE SURGICAL KIT PRODUCT COMPONENTS

Mounter Platform Short & Mounter Platform Long

Mounters are essential for extracting the implant from the container, placing it into the prepared bone socket, and finally tightening the implant. Mounters are fixed to the implant via screw fixation. They come with adapters for surgical handpieces (Mount Driver) and for a ratchet wrench (Ratchet Adaptor).

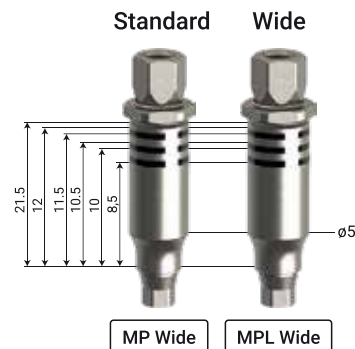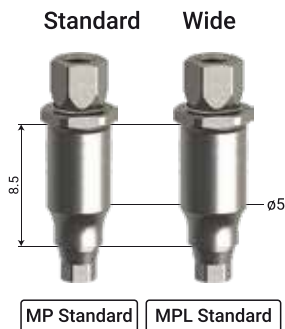

Bio3 GUIDE SURGICAL KIT

Bio3 GUIDE SURGICAL KIT PRODUCT COMPONENTS

Ratchet Adaptor is a wrench for connecting Mounter Platform with a ratchet wrench. GSK includes a short and a long adaptor.



Extractor Ex-M2.5 is used to unfasten implant from Mounter platform in case it is jammed.



Ex-M2.5

Pin for Template is a pin for fixing the navigation template to patient's jaw.



PPT2.0

Stop Ring H=2 mm a set of plastic locking rings 2 mm high, for adjusting the offset. Bio3 GSK includes 4 rings.



SR-H2 mm

Pin Drill D2 is a drill bit for preparing an opening in the bone when fixing the navigation template to patient's jaw.



TCB2.0-2.8



ADDITIONAL COMPONENTS (not included)

Sleeve for Template (not included in the standard GSK) is a guide bushing for the Pin Drill D2 and Pin for Template that is inserted into the navigation template. Sleeve for Template sets the direction for Pin Drill D2.



SFT2.0

Drill Guide DG-D5 and DG-D5-h3.5 (not included in the standard GSK) are guide bushings, which are inserted into the navigation template and set the direction for drills.



DG-D5-h3.5




DG-D5


Bio3 GUIDE SURGICAL KIT

TECHNICAL PARAMETERS OF DRILL GUIDES

Drill Guide DG-D5

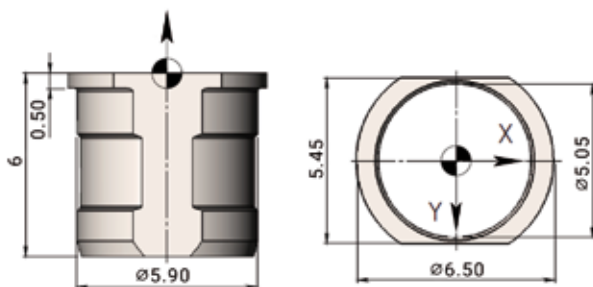

Drill Guide DG-D5-h3.5

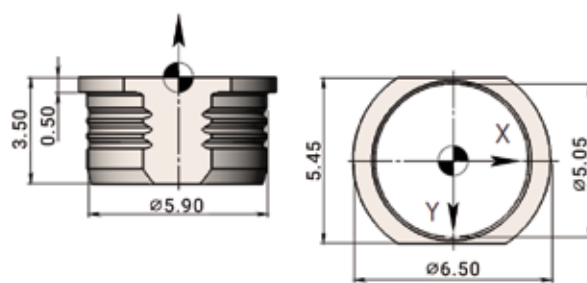

OFFSET SELECTION

Offset is the distance between the neck of the implant and the occlusal surface of the lip of the Drill Guide DG-D5 or Drill Guide DG-D5-h3.5.


Using Offset gives you the ability to adjust the drill position during the procedure for the desired result.

Note that 13 mm drills have an actual length of 15 mm, which can be compensated for using a stop ring or offset.


The maximum implant length that can be placed with this navigation kit is 13 mm and the diameter is 5.0 mm.

Drill Guide DG-D5 or Drill Guide DG-D5-h3.5 have variable Offset values. The table below shows the Offset values according to the implant lengths.

Implant length			
L = 6 mm	8.5	10.5	12.5
L = 8 mm	8.5	10.5	12
L = 10 mm	8.5	10	11.5
L = 11.5 mm	8.5	10	12
L = 13 mm	8.5	10.5	-

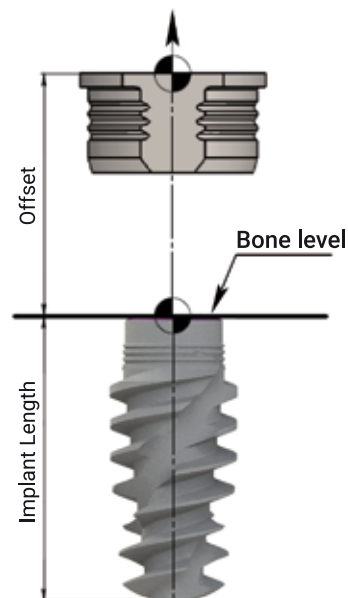

OFFSET SELECTION

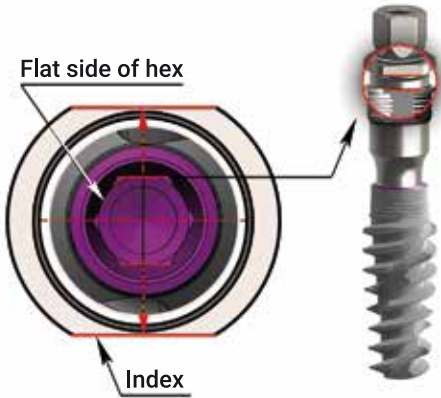

How to determine a rotation marker and ensure the correct orientation of the implant?

The flat side of the hex must be parallel to the flat side of the drill guide (index).

Mounter Platform Short

is used for implant placement, with a standard offset value of 8.5 mm.



Mounter Platform Long

is used for implant placement with an offset value more than 8.5 mm. It has laser markings for offset levels.



Depending on the selected offset value, different drill lengths are used. The table below shows the drill selection depending on the offset value and the implant length.

Implant length	Offset value, mm					
	8.5	10	10.5	11	11.5	12.5
L = 6 mm	Drill L8 (Ring 2 mm)	-	Drill L8	-	-	Drill L10
L = 8 mm	Drill L8	-	Drill L10	-	Drill L11,5	-
L = 10 mm	Drill L10	Drill L11.5	-	Drill L15 (Ring 2 mm)	-	-
L = 11.5 mm	Drill L11.5	Drill L15 (Ring 2 mm)	-	-	Drill L15	-
L = 13 mm	Drill L15 (Ring 2 mm)	-	Drill L15	-	-	-

Bio3 GUIDE SURGICAL KIT

DRILLING PROTOCOL

Protocol for placement of a 6 mm implant with offset 8.5 mm

Tissue Punch GS5/TP5.0
+ Drill Guide DG-D5-h3.5



Bone Mill BMG 4.8
+ Drill Guide DG-D5-h3.5



First Drill D1.9 L6
+ Stop ring SR-H2 mm
+ Drill Guide DG-D5-h3.5



Start Drill D1.9 L8
+ Stop ring SR-H2 mm
+ Drill Guide DG-D5-h3.5



Start Drill D2.8 L8
+ Stop ring SR-H2 mm
+ Drill Guide DG-D5-h3.5



Finish Drill D3.2 L8
+ Stop ring SR-H2 mm
+ Drill Guide DG-D5-h3.5



Finish Drill D3.6 L8
+ Stop ring SR-H2 mm
+ Drill Guide DG-D5-h3.5



Protocol for placement of a 11.5 mm implant with offset 8.5 mm

Tissue Punch
GS5/TP5.0
+ Drill Guide
DG-D5-h3.5



Bone Mill
BMG 4.8
+ Drill Guide
DG-D5-h3.5



First Drill
D1.9 L6
+ Drill Guide
DG-D5-h3.5



Start Drill
D1.9 L11.5
+ Drill Guide
DG-D5-h3.5



Start Drill
D2.8 L11.5
+ Drill Guide
DG-D5-h3.5



Finish Drill
D3.2 L11.5
+ Drill Guide
DG-D5-h3.5



PTI3.3/11.5
+ Mounter Platform MP Standard +
Drill Guide DG-D5-h3.5 + *Ratchet adaptor
SAD Hex.4 / **Mount Driver
MD Hex.4



*for Ratchet adaptor wrench **for hand piece

Finish Drill D4.0 L8
+ Stop ring SR-H2 mm
+ Drill Guide DG-D5-h3.5



PTI4.2/8
+ Mounter Platform MP Wide + Stop ring 2 mm SR-H2 mm
+ Drill Guide DG-D5-h3.5 + *Ratchet adaptor SAD Hex.4
/ **Mount Driver MD Hex.4



*for Ratchet adaptor wrench — **for hand piece

Finish Drill D4.8 L8
+ Stop ring SR-H2 mm
+ Drill Guide DG-D5-h3.5



PTI5.0/8
+ Mounter Platform MP Wide + Stop ring 2 mm SR-H2 mm
+ Drill Guide DG-D5-h3.5 + *Ratchet adaptor SAD Hex.4
/ **Mount Driver MD Hex.4



*for Ratchet adaptor wrench — **for hand piece

Finish Drill
D3.6 L11.5
+ Drill Guide
DG-D5-h3.5



PTI3.8/11.5
+ Mounter Platform MP Standard +
Drill Guide DG-D5-h3.5 + *Ratchet adaptor
SAD Hex.4 / **Mount Driver
MD Hex.4



*for Ratchet adaptor wrench — **for hand piece

Finish Drill
D4.0 L11.5
+ Drill Guide
DG-D5-h3.5



PTI4.2/11.5
+ Mounter Platform MP Standard +
Drill Guide DG-D5-h3.5 + *Ratchet adaptor
SAD Hex.4 / **Mount Driver
MD Hex.4



*for Ratchet adaptor wrench — **for hand piece

Finish Drill
D4.8 L11.5
+ Drill Guide
DG-D5-h3.5



PTI5.0/11.5
+ Mounter Platform MP Standard +
Drill Guide DG-D5-h3.5 + *Ratchet adaptor
SAD Hex.4 / **Mount Driver
MD Hex.4



*for Ratchet adaptor wrench — **for hand piece

Bio3 GUIDE SURGICAL KIT

DRILLING PROTOCOL

Protocol for placement of a 13 mm implant with offset 11.5 mm

Tissue Punch GS5/TP5.0
+ Drill Guide DG-D5-h3.5



Bone Mill BMG 4.8
+ Drill Guide DG-D5-h3.5



First Drill D1.9 L6
+ Stop ring SR-H2 mm
+ Drill Guide DG-D5-h3.5



Start Drill D1.9 L10
+ Drill Guide DG-D5-h3.5



Start Drill D1.9 L13
+ Stop ring SR-H2 mm
+ Drill Guide DG-D5-h3.5



Using Bio3 GSK you can install the following Bio3 implants:

ADVANCED IMPLANTS

Description	Code	D(mm)	L(mm)	Platform
Implant Advanced D3.3 mm L8 mm	ATI3.3/8	3.3	8	Standard
Implant Advanced D3.3 mm L10 mm	ATI3.3/10	3.3	10	Standard
Implant Advanced D3.3 mm L11.5 mm	ATI3.3/11.5	3.3	11.5	Standard
Implant Advanced D3.3 mm L13mm	ATI3.3/13	3.3	13	Standard
Implant Advanced D3.8 mm L8 mm	ATI3.8/8	3.8	8	Standard
Implant Advanced D3.8 mm L10 mm	ATI3.8/10	3.8	10	Standard
Implant Advanced D3.8 mm L11.5 mm	ATI3.8/11.5	3.8	11.5	Standard
Implant Advanced D3.8 mm L13mm	ATI3.8/13	3.8	13	Standard
Implant Advanced D4.2 mm L6 mm	ATIB4.2/6	4.2	6	Wide
Implant Advanced D4.2 mm L8 mm	ATIB4.2/8	4.2	8	Wide
Implant Advanced D4.2 mm L10 mm	ATIB4.2/10	4.2	10	Wide
Implant Advanced D4.2 mm L11.5 mm	ATIB4.2/11.5	4.2	11.5	Wide
Implant Advanced D4.2 mm L13mm	ATIB4.2/13	4.2	13	Wide
Implant Advanced D5 mm L6 mm	ATIB5/6	5	6	Wide
Implant Advanced D5 mm L8 mm	ATIB5/8	5	8	Wide
Implant Advanced D5 mm L10 mm	ATIB5/10	5	10	Wide
Implant Advanced D5 mm L11.5 mm	ATIB5/11.5	5	11.5	Wide
Implant Advanced D5 mm L13mm	ATIB5/13	5	13	Wide

PROGRESSIVE IMPLANTS

Description
Implant Progressive D3.3 mm L8 mm
Implant Progressive D3.3 mm L10 mm
Implant Progressive D3.3 mm L11.5 mm
Implant Progressive D3.3 mm L13mm
Implant Progressive D3.8 mm L8 mm
Implant Progressive D3.8 mm L10 mm
Implant Progressive D3.8 mm L11.5 mm
Implant Progressive D3.8 mm L13mm
Implant Progressive D4.2 mm L6 mm
Implant Progressive D4.2 mm L8 mm
Implant Progressive D4.2 mm L10 mm
Implant Progressive D4.2 mm L11.5 mm
Implant Progressive D4.2 mm L13mm
Implant Progressive D5 mm L6 mm
Implant Progressive D5 mm L8 mm
Implant Progressive D5 mm L10 mm
Implant Progressive D5 mm L11.5 mm
Implant Progressive D5 mm L13mm

Finish Drill D2.8 L10
+ Drill Guide DG-D5-h3.5



Finish Drill D2.8 L13
+ Stop ring SR-H2 mm
+ Drill Guide DG-D5-h3.5



Finish Drill D3.2 L13
+ Stop ring SR-H2 mm
+ Drill Guide DG-D5-h3.5



FIX3.5/13
+ Mounter Platform MP Standard + Drill Guide DG-D5-h3.5
+ *Ratchet adaptor SAD Hex.4 / **Mount Driver MD Hex.4



*for Ratchet adaptor wrench



**for hand piece

Code	D(mm)	L(mm)	Platform
PTI3.3/8	3.3	8	Standard
PTI3.3/10	3.3	10	Standard
PTI3.3/11.5	3.3	11.5	Standard
PTI3.3/13	3.3	13	Standard
PTI3.8/8	3.8	8	Standard
PTI3.8/10	3.8	10	Standard
PTI3.8/11.5	3.8	11.5	Standard
PTI3.8/13	3.8	13	Standard
PTIB4.2/6	4.2	6	Wide
PTIB4.2/8	4.2	8	Wide
PTIB4.2/10	4.2	10	Wide
PTIB4.2/11.5	4.2	11.5	Wide
PTIB4.2/13	4.2	13	Wide
PTIB5/6	5	6	Wide
PTIB5/8	5	8	Wide
PTIB5/10	5	10	Wide
PTIB5/11.5	5	11.5	Wide
PTIB5/13	5	13	Wide


FIX IMPLANTS

Description
Implant FIX D3.5 mm L8 mm
Implant FIX D3.5 mm L10 mm
Implant FIX D3.5 mm L11.5 mm
Implant FIX D3.5 mm L13mm
Implant FIX D4 mm L6 mm
Implant FIX D4 mm L8 mm
Implant FIX D4 mm L10 mm
Implant FIX D4 mm L11.5 mm
Implant FIX D4 mm L13mm
Implant FIX D4.5 mm L6 mm
Implant FIX D4.5 mm L8 mm
Implant FIX D4.5 mm L10 mm
Implant FIX D4.5 mm L11.5 mm
Implant FIX D4.5 mm L13mm
Implant FIX D5 mm L6 mm
Implant FIX D5 mm L8 mm
Implant FIX D5 mm L10 mm
Implant FIX D5 mm L11.5 mm
Implant FIX D5 mm L13mm

Code	D(mm)	L(mm)	Platform
FIX3.5/8	3.5	8	Standard
FIX3.5/10	3.5	10	Standard
FIX3.5/11.5	3.5	11.5	Standard
FIX3.5/13	3.5	13	Standard
FIX4.0/6	4	6	Standard
FIX4.0/8	4	8	Standard
FIX4.0/10	4	10	Standard
FIX4.0/11.5	4	11.5	Standard
FIX4.0/13	4	13	Standard
FIX4.5/6	4.5	6	Standard
FIX4.5/8	4.5	8	Standard
FIX4.5/10	4.5	10	Standard
FIX4.5/11.5	4.5	11.5	Standard
FIX4.5/13	4.5	13	Standard
FIX5.0/6	5	6	Standard
FIX508	5	8	Standard
FIX5.0/10	5	10	Standard
FIX5.0/11.5	5	11.5	Standard
FIX5.0/13	5	13	Standard

Bio3 GUIDE SURGICAL KIT

TOOLS INSTRUCTION FOR USE

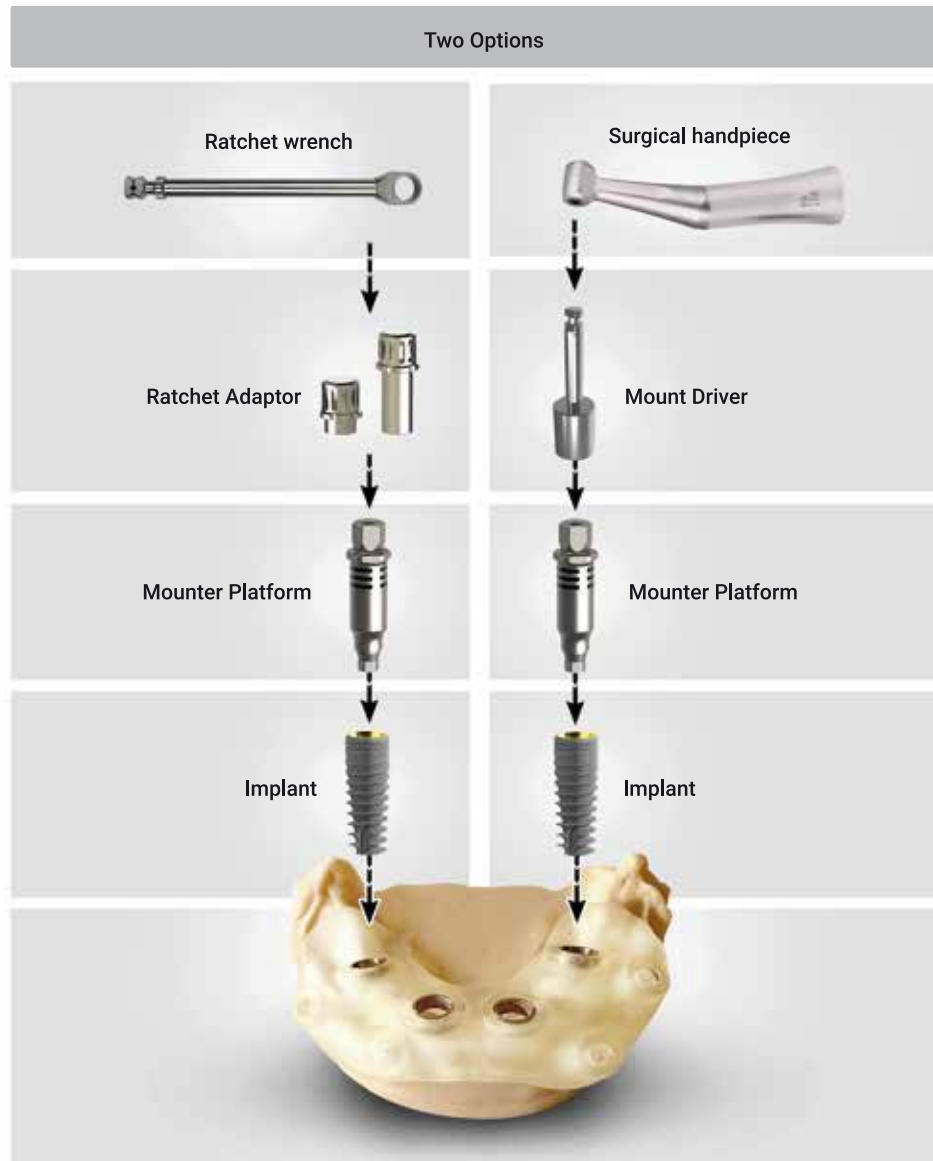



NOTES

NOTES

[illegible]



Version GSK.EN.001.2026 CE



CONTACT US:



Bio3 Implants GmbH
Hanauer Str. 1-5, 75181,
Pforzheim, Deutschland



+49 723 160 84 104



www.bio3-implants.com



info@bio3-implants.com

Follow us:

